


How it works


1. Identify and mark

Identify cows with clinical mastitis, mark the quarter(s), collect a milk sample, and treat with KetoMax 15%

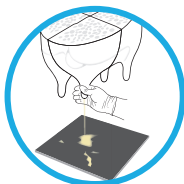


2. Load cartridge

Add milk to Mastatest cartridge, then replace lid

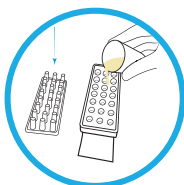

3. Load lapbox and run Mastatest

Load the sample into the lapbox, then follow the on-screen prompts to start the test

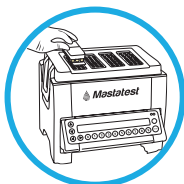

4. Get results

Fully interpreted report sent to you and your vet in 24 hours



5. Treat with confidence

Use the test results to follow your vet's personalised treatment plan*



*Treatment plan needs to be set-up by your veterinarian

Talk to your vet today about about the benefits of Mastatest for your farm



Now with the G3 lapbox: Future-proof your mastitis management

Find out more about the new generation of Mastatest
– scan the QR code to see the differences:

Lapbox	\$ _____
Clinical test	\$ _____
Subclinical test (HiSCC)	\$ _____

Scan the QR code
to see how easy
Mastatest is to use



 **Mastatest**

Take the guesswork out of mastitis treatment



FAST



EASY



INFORMED
DECISIONS

Mastitis costs you time, money and lost production.
Mastatest gives you clear, actionable results in just
24 hours – so you can treat with confidence.

Find the bug. Use the right drugs

KetoMax 15% ACVM Registration No. A011031. Restricted Veterinary Medicine.
Available only under veterinary authorisation.



www.agrihealth.co.nz

0800 821 421

AgriHealth

MSTT-DLE-2025

AgriHealth

Why farmers trust Mastatest

Fast and easy

- ✓ Set-up in minutes – load a sample and go
- ✓ Results in 24 hours – sent straight to you and your vet

Saves you money

- ✓ 1 in 5 clinical mastitis cows might not need antibiotic treatment. Find those cows, treat with Ketomax 15% and save antibiotics
- ✓ Less milk withhold means more milk in the vat

Take away the guess work

- ✓ Vet-supported treatment plans – work with your vet to personalise individual cow treatments for your herd
- ✓ Confident decisions – no more guessing or ‘trial-and-error’ treatments

Which Mastatest cartridge do you need?

Mastatest offers two diagnostic options to help you make the right treatment decisions

Clinical Tests (NZP) For Mastitis Cases



- ✓ Test cows with clinical mastitis to identify the bacteria causing infection
- ✓ Get targeted treatment recommendations in just 24 hours
- ✓ Reduce unnecessary antibiotic use and improve cure rates

Subclinical Tests (HiSCC) For High SCC Cows



- ✓ Test RMT-positive cows or high SCC cows from herd tests
- ✓ Identify bacteria early to reduce infection spread, lower bulk SCC, and unlock milk premiums
- ✓ Make informed management, culling and dry-off decisions

What farmers are saying:

“You’d struggle to find a better return on investment than Mastatest.”

Andrew Lord, Waikato dairy farmer

“Just test every clinical case, and do whatever Mastatest recommends – there’s no more guessing.”

Marty Neill, Waikato dairy farmer

